

PARI Export Service Center



0049 (0) 8151 279-220

Call...



... for questions on **use, handling, cleaning or maintenance** of your **PARI** nebuliser or inhalation device.



... for information about **spare parts of today's and yesterday's PARI products** and/or about where to order these.



... for additional information or individual questions about the entire **PARI** product range.

local distributor



PARI GmbH
Spezialisten für effektive Inhalation
Moosstrasse 3 · D-82319 Starnberg
Telefon +49 (0) 8151 279-0
Fax +49 (0) 8151 279-101



You can find **PARI** in the Internet:
www.pari.de
E-Mail: info@pari.de

Specialists in effective inhalation



PARI PEP® Systems

Time-saving combination
of physiotherapy and
inhalation treatment



- Mobilises secretions and thus facilitates clearing of the lungs
- Stabilises the respiratory tract by adjustable expiration resistance: counteracts bronchial collapse
- Pressure display for setting expiration resistance and for monitoring during PEP treatment (PARI PEP® System II)
- Can also be used without the nebuliser in physiotherapy

©2003 PARI GmbH Spezialisten für effektive Inhalation 018D0009 07/2007

Specialists in effective inhalation



PARI PEP® Systems

Brings movement to your airways

The PARI PEP® System provides an effective combination of PEP (Positive Expiratory Pressure) and inhalation treatment for secretion clearance in acute and chronic diseases of the lower respiratory tract. When the respiratory tract is unstable, troublesome secretion cannot be cleared because the increased pressure caused by coughing constricts the bronchi. The resistance that is created by the PARI PEP® System during the expiratory phase prevents the respiratory tract from collapsing. The peripheral airways remain open longer, enabling a greater quantity of medication to reach its target site during inhalation. As a result, it is easier to cough up secretion and more air is exhaled.



PARI PEP® System I ①
for **PARI LC PLUS®**,
PARI LC STAR® and
PARI LL Nebuliser.
Without manometer
Item No. 018G6101

PARI PEP® System II ①
for **PARI LC PLUS®**,
PARI LC STAR® and
PARI LL Nebuliser.
With manometer
Item No. 018G6200

PARI PEP® S System for ②
PARI Nebuliser (except PARI LC
SPRINT BABY Nebuliser, PARI
BABY Nebuliser and PARI TIA
Nebuliser S)
Item No. 018G4000

Accessory:
Manometer
0-100 mbar, including
Pressure Hose
Item No. 018B1600

